


HAND CONTROLLER MANUAL FOR LOVESEAT DUAL MOTOR LIFT CHAIR

Please read the hand controller manual carefully, before starting to use the loveseat dual motor lift chair.

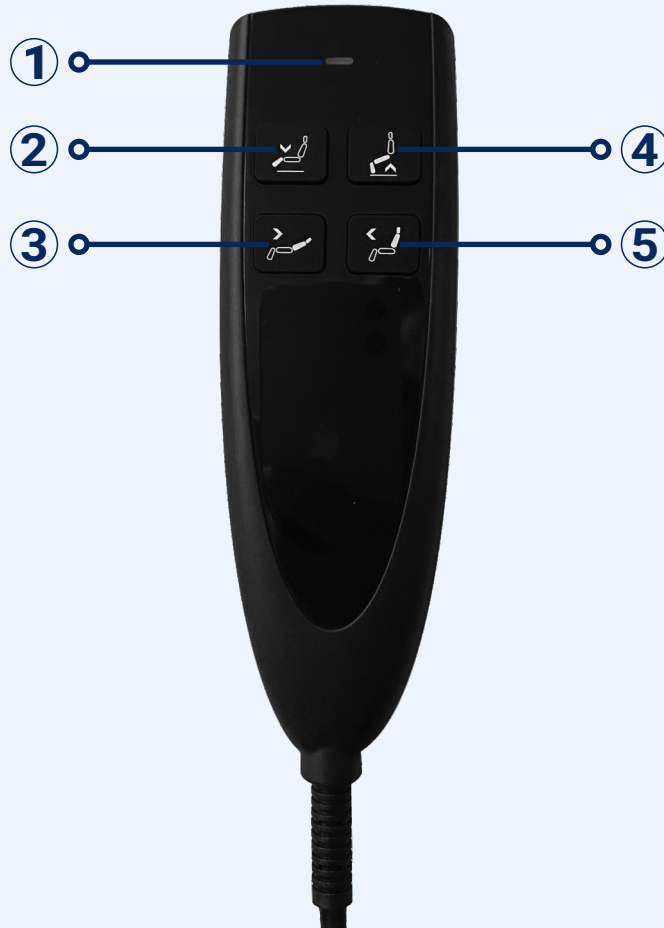

A dual motor lift chair has independent controls for the back and leg rest, meaning they can be moved independently.

①

- A. Power LED.
- B. Light will turn ON when power is connected.

④

- A. Raise the chair to standing from seated.
- B. Lower the footrest.

②

- A. Lower the chair from standing position.
- B. Raise the footrest.

⑤

- Raise the backrest from recliner position.

③


- Lower the backseat from seated position.

SAFETY INSTRUCTION:

- *Please ensure the chair is in an upright seated position before lifting.
- *Ensure there is enough room between wall and chair before reclining.

HAND CONTROLLER MANUAL FOR ALTON DUAL MOTOR LIFT CHAIR

Please read the hand controller manual carefully, before starting to use the Alton dual motor lift chair.

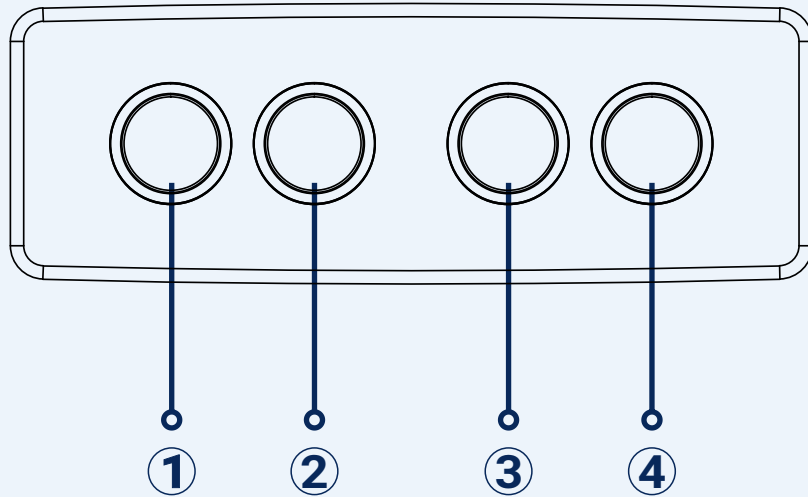

Operating Instruction:

①

Returning backrest to upright position.

②

Reclining backrest.

③

A. Returning footrest to seating position.
B. Lift chair to standing position.

④

A. Opening footrest.
B. Lower chair to seated position.


SAFETY INSTRUCTION:

*Please ensure the chair is in an upright seated position before lifting.

*Ensure there is enough room between wall and chair before reclining.

HAND CONTROLLER MANUAL FOR DUAL MOTOR LIFT CHAIR

Please read the hand controller manual carefully, before starting to use the dual motor lift chair.

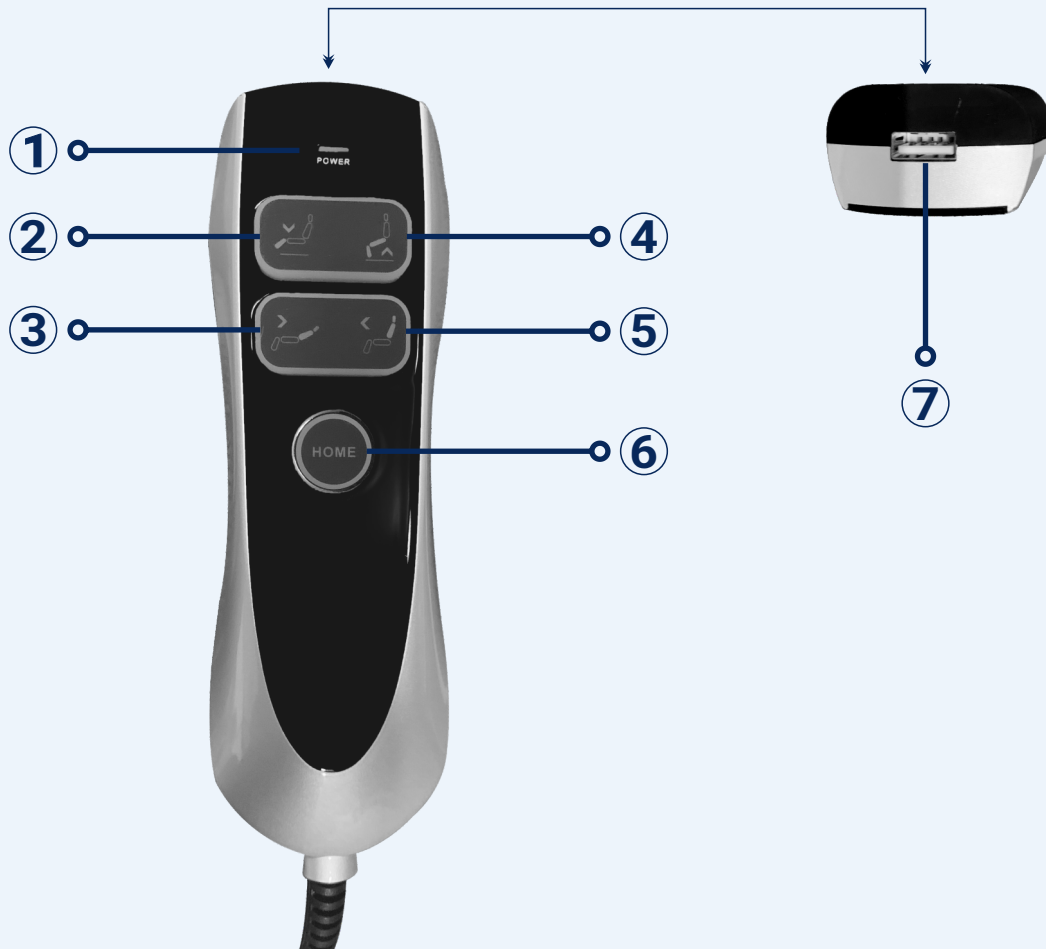

A dual motor lift chair has independent controls for the back and leg rest, meaning they can be moved independently.

1

- A. Power LED.
- B. Light will turn ON when power is connected.

2

- A. Footrest up.
- B. Lower back to seated.

3

- Recline backrest.

4

- A. Footrest down.
- B. Lift chair to standing.

5

- Lift backrest.

6

- A. Home button.
- Lowers chair back to original seated position from lifted position.
- B. When press and hold this button, following function will move simultaneously till you leave your fingertip from the button.
 - Backrest will raise from the button.
 - Footrest will close.

7


- A. USB charger with output current up to 1.0 A.

SAFETY INSTRUCTION:

- *Please ensure the chair is in an upright seated position before lifting.
- *Ensure there is enough room between wall and chair before reclining.

HAND CONTROLLER MANUAL FOR ECCLES DUAL MOTOR LIFT CHAIR

Please read the hand controller manual carefully, before starting to use the Eccles dual motor lift chair.

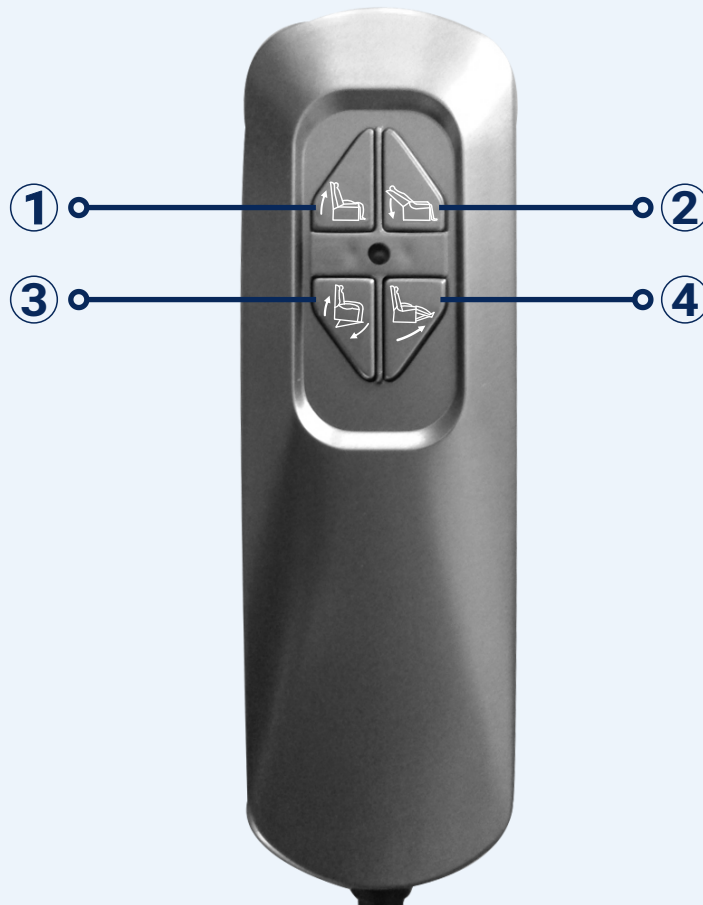

Operating Instruction:

①

Returning backrest to upright position.

②

Reclining backrest.

③

A. Returning footrest to seated position.
B. Lift chair to standing position.

④

A. Opening footrest.
B. Lower chair to seated position.


SAFETY INSTRUCTION:

*Please ensure the chair is in an upright seated position before lifting.

*Ensure there is enough room between wall and chair before reclining.

HAND CONTROLLER MANUAL FOR HAMMEL AND LIVINGSTONE DUAL MOTOR LIFT CHAIR

Please read the hand controller manual carefully, before starting to use the dual motor lift chair.

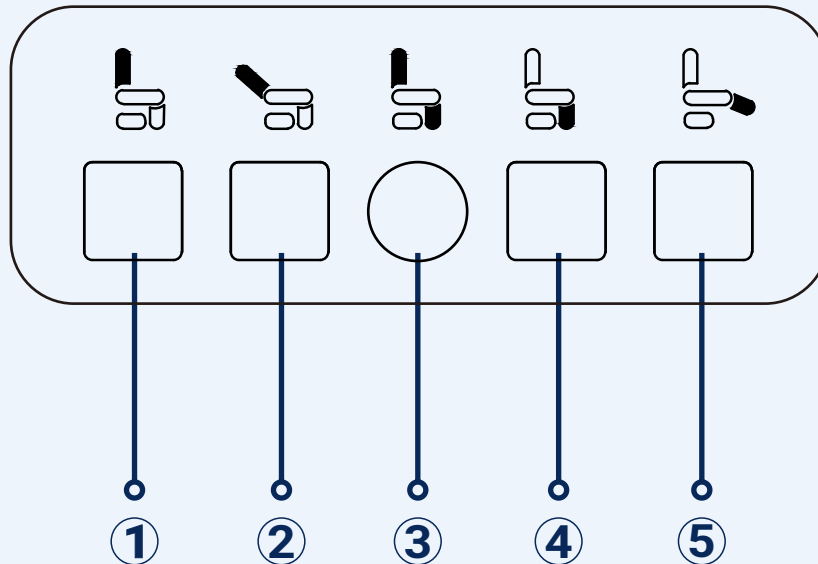

Operating Instruction:

①

Returning backrest to upright position.

②

Reclining backrest.

③

Home button.
Lift headrest and lower chair to seated position.

④

A. Returning footrest to seating position.
B. Lift chair to standing position.

⑤

A. Opening footrest.
B. Lower chair to seated position.


SAFETY INSTRUCTION:

*Please ensure the chair is in an upright seated position before lifting.

*Ensure there is enough room between wall and chair before reclining.

HAND CONTROLLER MANUAL FOR MESSAGE DUAL MOTOR LIFT CHAIR

Please read the hand controller manual carefully, before starting to use the message dual motor lift chair.

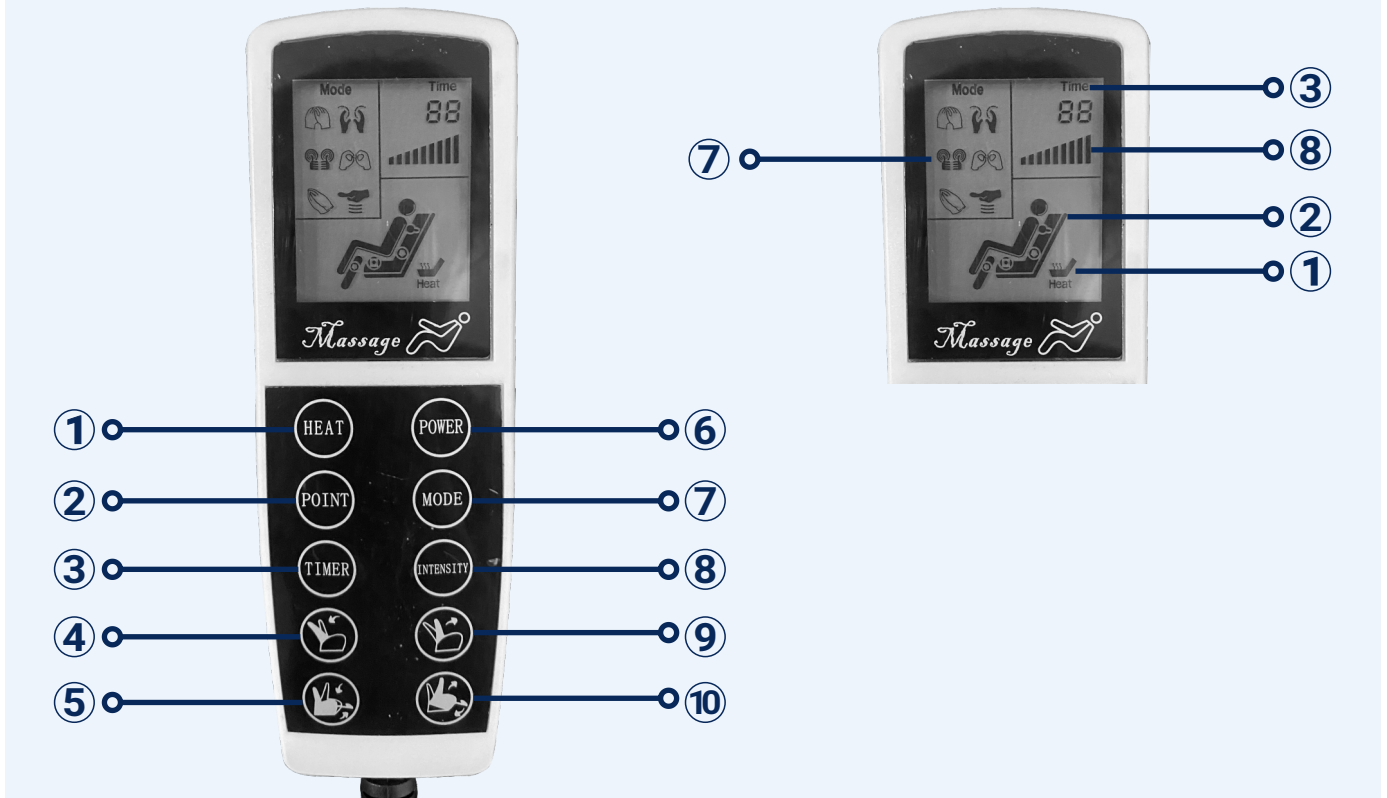

A dual motor lift chair has independent controls for the back and leg rest, meaning they can be moved independently.

1

HEAT: Heating ON and OFF key.
Heat symbol will display when ON.

2

POINT: 4 massage points selection key.
Small symbol will display when ON & on specific point of body.

3

TIMER: Massage timing selection key, range from 5 mins to 30 mins.

4

Recline button. Lower the backrest.

5

A. Lower the chair from standing.
B. Raise the footrest.

6

POWER: Turn ON and OFF selection key.
Screen display ON when OFF.

7

MODE: 6 working modes selection key. Each mode symbol will display individually when ON.

8

INTENSITY: Choose massage intensity - Soft, Moderate, Strong.

9

Raise the backrest from recliner position.

10


A. Raise the chair to standing position.
B. Lower the footrest.

SAFETY INSTRUCTION:

- *Please ensure the chair is in an upright seated position before lifting.
- *Ensure there is enough room between wall and chair before reclining.

HAND CONTROLLER MANUAL FOR SINGLE MOTOR LIFT CHAIR

Please read the hand controller manual carefully, before starting to use the single motor lift chair.

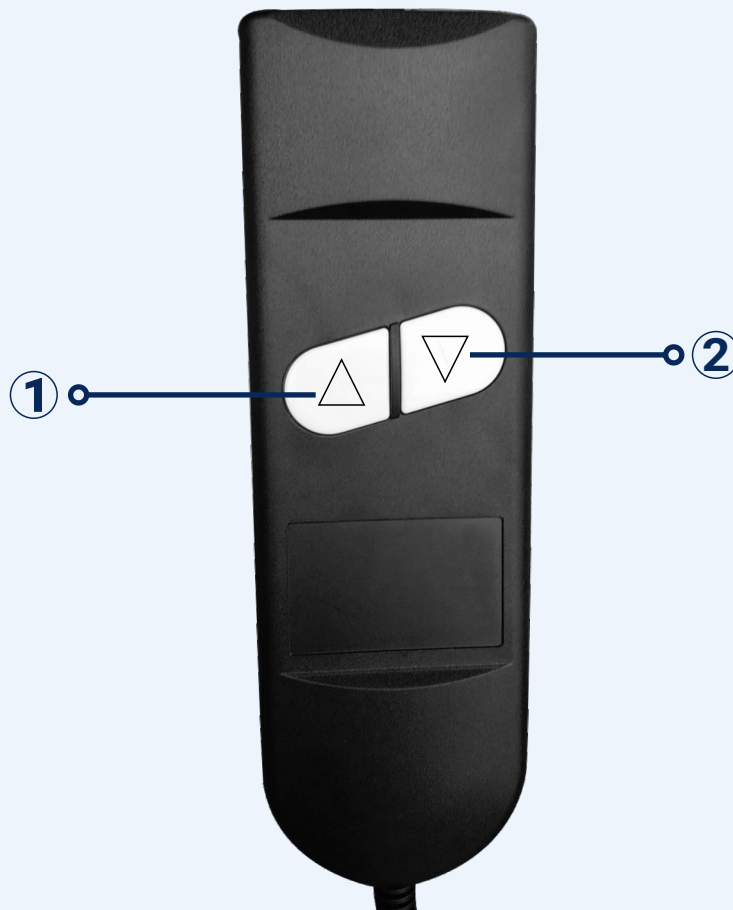

A single motor lift chair has one operation panel that controls both the back recliner and footrest and lift simultaneously.

①

- A. Raise backrest and lower footrest from recliner position.
- B. Raise the chair from seated position.

②

- A. Lower the chair from standing position.
- B. Recline and raise footrest from seated position.

SAFETY INSTRUCTION:

- *Please ensure the chair is in an upright seated position before lifting.
- *Ensure there is enough room between wall and chair before reclining.